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## Usage instructions

### For patients and orthopaedics specialists

Dear customers,

Thank you for your confidence in us. You have made a good decision with your purchase. It is our intent and goal to work in close collaboration with our partners, doctors, medical insurers and health services, to ensure our shoes provide the best possible care for the feet. Find out more about our products and visit our website:

[www.mygeneration-schein.de](http://www.mygeneration-schein.de)

Before using our **medical product**, please read the instructions for use through carefully and take note of the following instructions, in particular the safety information:

#### 1. Intended/indicated purpose:

**Orthotic shoe:** - Shoes over leg orthoses

- To allow and maintain a suitable walking function (in connection with an orthosis)
- If an orthosis with a foot component is to be worn long-term on the foot and the orthosis cannot be combined with standard ready-made shoes

**Stable shoes:** - Shoes for paralytic conditions

- Support and guidance for the ankles
- To allow and maintain a suitable walking function
- Impairments to mobility (walking) e.g. in case of paralytic conditions with a spastic component tending towards drop foot
- Impairments to mobility (here walking) e.g. in case of severe damage to the ankle ligaments over the course of conservative or post-operative treatment or in case of spastic paralysis (after a cerebral insult or infantile cerebral palsy) as a temporary immediate treatment

**Corrective shoes:** - Anti-varus / neutral shoes to safeguard the success of a corrective procedure

- To ensure the results of treatment are maintained and allow/retain a suitable walking function.
- Impairments to mobility (walking) caused by damage to the foot position (sickled feet) and corrective procedures carried out at pre-school age, after the start of the walking phase.

#### 2. Location/conditions of use::

- Our products are conceived to be worn outdoors but can also be worn at home if needed. One exception is the slippers, which should exclusively be worn in the house.
- For hygienic reasons, we recommend 2 pairs of shoes. One for indoors, one for out..

**Please take note that wet or smooth floors, as well as snow and ice, can raise the risk of slipping. Particular care is advisable in this regard.**

#### 3. Instructions on use:

- Open and undo the laces or closures to put the shoe on. Put on the shoe and tie the laces / do up the

closures.

- Always put the shoe on the correct foot. The shoes are labelled on the interior with "R" (right) and "L" (left).
- It is easiest to put on the MyGeneration shoes with a shoe horn.
- The use of a medical product is solely intended for a single patient. Reuse is not permitted.
- The way the anti-varus shoes are constructed can flatten out the arch of the foot. We recommend counteracting this with a suitable orthopaedic insert.
- Patients requiring stable shoes should be given suitable inserts depending on the form of their condition.
- In case of single-side orthotics, it is sensible to achieve static balance on the unaffected side using an insert or insole.
- The duration and frequency of use will be determined by your doctor.
- We recommend changing the shoes after a day's use.

**Please take note of our instructions before wearing the "MyGeneration" shoes, and particularly before the first use.**

#### 4. Safety notices and usage risks:

- If bruising, pain or other negative manifestations occur, you should visit your doctor or supplier without delay.
- Before wearing, feel the interior of the shoes and remove any foreign bodies.
- To avoid chafing, never wear the shoes without socks.
- Only use the shoes if the skin of the feet is unimpaired.
- You should check your feet daily for redness and bruises.
- Wear new shoes for short durations initially and check the feet for bruising after approx. 20 minutes.
- To avoid bruises and circulation problems, do not tie/strap the shoes too tight. Check the closure directly and adjust it as needed.
- Loose shoes can lead to tripping or bruises/chafing caused by slippage.

#### Important note:

**In case of problems with your feet or shoes, please contact your doctor or the specialist supplier of your orthopaedic shoes immediately.**

#### 5. Contraindications:

- If you have existing skin allergies to the materials used (see point 6), please talk to your doctor before use.
- In case of extensive foot deformities, a ready-made MyGeneration shoe is not suitable.

#### 6. Material information:

MyGeneration shoes use various **smooth** and **suede leathers, textiles** and **used-look** textiles.

- **Upper leather:** Shaft material – smooth and suede leather, textiles  
(The textiles used depend on the model – in use are fleece, mesh, canvas, microfibre, denim and felt)

- **Padding material:** Microfibre, calfskin, Sympatex, warm lining and textile
- **Footbed / insert:** Polyurethane with textile covering Or Texon insert with laminated multiform cover Or leather cover
- **Sole:** Polyurethane, ethylene vinyl acetate
- **Further materials:** Polyamide, plastic and metal

## 7. Cleaning and care information:

### Before first use

- Before putting the shoes on for the first time, spray the upper material with a standard moisture protection spray. This will protect against the wet.
- Repeat the process again after extensive use.

### Cleaning

- In case of soiling, often a simple brushing of the affected parts is sufficient. For this, in the case of smooth leather and textile upper materials, we recommend the use of a stiff brush.
- For all suede and microfibre materials, the use of a nubuck brush is required.

### Disinfection

- The padding material can be cleaned with a damp cloth. We recommend a suitable disinfectant to clean the padding (you can get this e.g. in a chemist's).
- For the upper leather, no disinfectants may be used. Please do not wash the MyGeneration shoe in the washing machine.

### Leather care

- Small scratches or scarified insect bites in leather are a sign of its natural origins. To keep the leather supple, nourishing conditioner should be regularly applied to the shoes. You can find suitable creams or nubuck care sticks in specialist shops.
- Different leathers need the corresponding shoe care treatments.

### Drying

- Never place wet or damp shoes in the direct proximity of a heat source to dry out.
- Please avoid using radiators and direct sunlight to dry shoes.
- It is best to stuff wet or damp shoes with newspaper or use a suitable shoe tree.
- Give your shoes time to dry. This can take up to two days, depending in the level of moisture penetration.

### Storage

- Please store your shoes dry and at room temperature.

**You can find extensive care and cleaning tips on our home page.**

## 8. Disposal:

- The disposal of products and packaging must take place according to legal requirements.

## 9. General information / maintenance:

- The lifespan and durability of the shoe, assuming proper use, treatment and care, is three years.
- Please take note of our care information.
- As children are always growing, we ask you to assess the fit of the shoe frequently or set up regular check-

ups with your orthopaedic specialist.

- Check the shoe regularly for functionality, wear or damage to closures, soles etc.

## 10. Assembly instructions:

- Remove the shoe filler (e.g. paper) and adjust the shoe to fit.
- If necessary, adjust the upper material (e.g. correct lace, strap lengths)
- In case of individual alterations to the product (e.g. sole stiffening or rolling), we would like to draw your attention to the fact that the aid must be permanently marked with the label "custom-made product".

## 11. Warranty:

- If proper use is assured, we offer the legally required warranty.

Please inform schein orthopädie service KG in case of abnormalities, problems or deficiencies to the shoes.

We wish you great pleasure in wearing our shoes.

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